



Care you precisely



SK-LLD05E

LED OPERATION LAMP

www.sksurgitech.com

SKSurgitech™
by Saikang Medical



SK-LLD05E

FEATURES AND HIGHLIGHTS

The LED surgical shadowless lamp is composed of multiple lamp bulbs. The bulb has a longer life than the traditional shadowless lamp. It is round or petal-shaped. It is fixed on the balance arm suspension system. It is stable in positioning and can be moved vertically or cyclically. It can meet different heights during surgery. The petal-shaped surgical shadowless lamp is also a kind of LED. It is introduced separately because the lamp cap is petal-shaped and has a special shape.



Technical parameters

▪ Diameter of lighthead (mm):	860
▪ Illumination at 1m (Lux):	140000
▪ Brightness control range:	10-100% stepless adjustment
▪ Color temperature (Kelvin):	3500-5000
▪ Color rendering index (CRI):	≥96
▪ Temperature rise (Surgeon head)(°C):	≤3
▪ Temperature rise (Surgical field)(°C):	≤1
▪ Total irradiance Ee (W/m2):	≤130
▪ Radiant power Ee/Ec (mW/m2lux):	≤4
▪ Effective focusing depth (cm):	100
▪ Lighting spot size (cm):	22-31
▪ L1+L2 depth of illumination (cm):	100
▪ Bulb type:	LED

▪ Bulb brand:	OSRAM
▪ Bulb quantity:	185
▪ Bulb power (w):	1
▪ Average bulb life (hours):	≥50000

Technical configuration

<input checked="" type="checkbox"/> Rotating arm	1 set
<input checked="" type="checkbox"/> Balance arm	1 set
<input checked="" type="checkbox"/> C-arm	1 set
<input checked="" type="checkbox"/> Handle	1 pcs


Material technology

The coating is sprayed with world-leading Akzo Nobel powder coating and anti-glare treatment, which meets the demanding requirements of surgical lighting. The design of the lamp holder focuses on reducing the turbulence. The streamlined lampshade of the whole lamp arm is carefully designed according to the principle of aerodynamics, which is fully in line with The standard of a modern new operating room.




Bulb

The bulbs are imported from Germany OSRAM brand, using a special optical system, 360° uniformly illuminated on the observed object, no shadows, true reproduction of the organization color, and achieve constant color temperature under any light intensity conditions. The material is hard and no mercury pollution. Also, the emitted light does not contain infrared and ultraviolet radiation pollution.

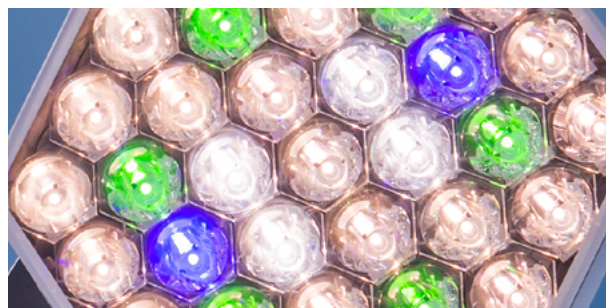

Color temperature&illumination

Using a new type of LED cold light source, the illuminance can reach 140,000 Lux, which can realize 16-level brightness adjustment; the color temperature is in the range of 3500-5000K, and four-level color temperature adjustment can be achieved. While meeting the high illuminance, adjust the lighting parameters according to the needs of different surgeons, so that the surgeon's perception of light is soft and no glint.

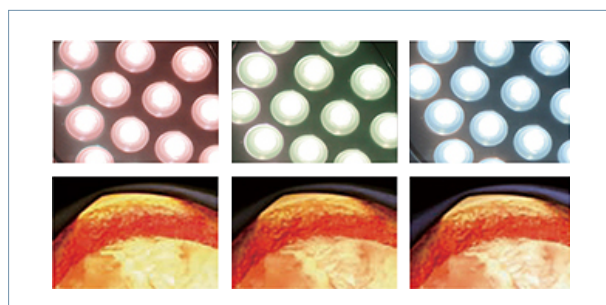


Color rendering index

The color rendering index reaches 96, which truly reflects the color of human tissues. It is suitable for various surgical scenes and greatly reduces the visual fatigue of medical staff due to long-term operations.



Temperature rise (surgical field)

LED cold light source has no infrared radiation, coupled with our nano-coated radiator, created our excellent heat dissipation effect. It solves the problem of the doctor's head temperature being too high during the operation, and creates a more comfortable operation environment for the doctor; it also effectively avoids the loss of water and infection of the surgical wound, which is conducive to the rapid recovery of the patient's wound after the operation.

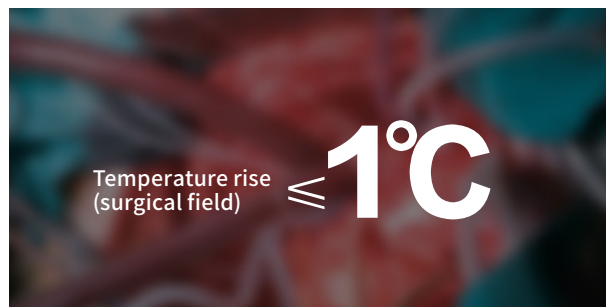

Lens

Each lens adopts a 40mm super large lens, and the lens arrangement is arranged according to the biology of insect compound eyes, and the mutual distance is less than 1mm, so that the shadowless effect is better.



Operating handle

The detachable disinfection handle is made of high-strength ABS, which can be disinfected and sterilized at a high temperature above 135°, ensuring the durability and reliability of the handle.



Other functions

With a variety of lighting modes, one-key switching, adapt to different clinical surgery scenes.





 **CONSTANT
INNOVATION**



Design



Research



Manufacturing



Maintenance



Service

 **WORLDWIDE
BRANCHES**

USA

Saikang Medical Technology, Inc.
17800 Castleton St Ste 180,
City of Industry, 91748, California.
Tel.: +1 949648 2776

United Kingdom

UK Healthward International Ltd.
Dept. 111, 196 High Road Wood
Green, N22 8HH, London.
Tel.: +44 2032399738

Russia

Saikang Medical Russia LLC
Truda 174,
454080 Chelyabinsk.
Tel.: +7 (351) 214 5566 / 8811 / 1516

East Africa





Saikang Medical Solutions Ltd.
Viraj Complex Unit 3, Mombasa
Road, P.O. Box 14262-00800, Nairobi,
Kenya.
Tel.: +254 733 704 690

Middle East

Saikang Medical Supplies LLC
X22-S03 England Cluster,
International City, Dubai.
United Arab Emirates.
Tel.: +20 128 8962497

SKSurgitech™
by Saikang Medical

Jiangsu Saikang Medical Equipment Co., Ltd.
STOCK CODE: 870098

 No. 35, Lehong Road · Zhangjiagang (Jiangsu) · China
 +86 18021231901 ·  24/7
 export@sksurgitech.com
 www.sksurgitech.com



Company video



General catalogue